

# Head and Stem Combinations: Unipolar and Bipolar Femoral Heads

- Indicates "Combination is approved by Zimmer, Inc."
- Indicates "Combination is not approved by Zimmer, Inc."

	<b>Unipolar Heads:</b>						<b>Bipolar Heads<sup>1</sup>:</b>							
	Unipolar Head (4510-XX-XXX) (12/14 taper)	Adapter for Unipolar Head (4510-XX-XXX) (12/14 taper)	Endo Femoral Head (6 degree)	Endo Femoral Head Adapter (6 degree)	Unipolar Head (non-modular) (30.38/60.09) (12/14 taper)	VerSys Endo Femoral Head (12/14 taper)	VerSys Endo Femoral Head Adapter (12/14 taper)	Bipolar Head (AP) (1946/1989)	Bi-polar head (4404-22/26-XXX)	Continuum Bipolar Head	Cupola Mobile	Cupule Mobile Retentive	Durasul Bi-polar head	Modular-bipolar head
ACA stem														
Acora cemented stem														
ALLOCLASSIC SLA and SL-64 stem <sup>2</sup>	✓	✓			✓		✓	✓		✓	✓	✓	✓	
ALLOCLASSIC SLL-Revision stem	✓	✓			✓		✓	✓		✓	✓	✓	✓	
ALLOCLASSIC stem <sup>2</sup>	✓	✓			✓		✓	✓		✓	✓	✓	✓	
ALLOCLASSIC VARIALL SLV stem					✓									
ALLOCOR AN stem					✓									
Anatomic stem			✓	✓										
Anatomic II stem						✓	✓							
Anatomique uncemented HA stem					✓									
Apollo Hip System	✓	✓			✓		✓	✓		✓	✓	✓	✓	
APR Hip System	✓	✓			✓		✓	✓		✓	✓	✓	✓	
APS Natural-Hip System						✓	✓							
Avenir Müller stem	✓	✓			✓	✓	✓	✓		✓	✓	✓	✓	
Avenir Cemented stem	✓	✓			✓	✓	✓	✓		✓	✓	✓	✓	
BEO stem					✓									
CDH stem PROTASUL-S30 (monoblock)														
CLS SPOTORNO stem	✓	✓			✓		✓	✓		✓	✓	✓	✓	
Concept cemented stem														
Continuum Cobrex stem									✓					
Continuum F-115 Hip stem									✓					
Continuum F-117 Collarless Hip stem									✓					
Continuum J-Taper stem									✓					
Continuum Proxilock stem (porous and HA coated)									✓					
CPS uncemented stem					✓		✓	✓		✓	✓	✓	✓	
CPT (6 Deg, CoCr) stem			✓	✓				✓				✓	✓	✓
CPT (6 Deg, SST) stem								✓						✓
CPT 12/14 CoCr stem						✓	✓	✓				✓	✓	✓
DCM-J Natural - Hip System						✓	✓	✓				✓	✓	✓
DSP / TPP / PDC					✓									
Emeraude stem					✓									
Epoch stem						✓	✓	✓				✓	✓	✓
ESKA 8/10 stem														
ESKA 12/14 stem														
Etoile cemented stem														
EXAFIT Primary stem														
EXAFIT Revision stem														
Fitmore Hip Stem	✓	✓			✓		✓	✓		✓	✓	✓	✓	
FracSure Hip system	✓	✓			✓		✓	✓		✓	✓	✓	✓	
Heritage stem (XX-8095-XXX-XX)			✓	✓								✓	✓	✓
Intermedia stem					✓							✓	✓	
Lamella stem					✓							✓	✓	
M/L Taper stem						✓	✓	✓				✓	✓	✓
M/L Taper stem with Kinectiv Technology														
MARBURG stem					✓									
Mayo 12/14 stem						✓	✓	✓				✓	✓	✓
Mayo 6 Deg stem			✓	✓								✓	✓	✓
Mercurie/F23 cementless stem						✓	✓	✓				✓	✓	✓

1. See "Articulating Combinations" and "Head and Stem Combinations"  
 2. ALLOCLASSIC SLA and SL-64 (all sizes) and ALLOCLASSIC stems (sizes 01-6) can only be used with Bipolar heads (4404-22/26-XXX, 4405-22/26-XXX) up to size 56.

Continued on Page 2

# Head and Stem Combinations: Unipolar and Bipolar Femoral Heads

- Indicates "Combination is approved by Zimmer, Inc."
- Indicates "Combination is not approved by Zimmer, Inc."

	Unipolar Heads:							Bipolar Heads <sup>1</sup> :							
	Unipolar Head (4510-XX-XXX) (12/14 taper)	Adapter for Unipolar Head (4510-XX-XXX) (12/14 taper)	Endo Femoral Head (6 degree)	Endo Femoral Head Adapter (6 degree)	Unipolar Head (non-modular) (30.38/60.09) (12/14 taper)	VerSys Endo Femoral Head (12/14 taper)	VerSys Endo Femoral Head Adapter (12/14 taper)	Bipolar Head (AP) (1946/1989)	Bi-polar head (4404-22/26-XXX)	Continuum Bipolar Head	Cupola Mobile	Cupule Mobile Retentive	Durasul Bi-polar head	Modular-bipolar head	Multipolar Bipolar Cup
Mercur/F23 cemented stem															
METABLOC stem															
Minima cemented stem															
Modular Austin Moore stem															
Modular Calcar Replacement stem															
Moore monoblock stem															
MOST Options system															
MS-30 stem															
Muller stem (XX-8050-051/056-XX)															
Müller Straight stem PROTASUL-10															
Müller Straight stem PROTASUL-S30															
Müller Straight stem PROTASUL-S30 (monoblock)															
Multilock stem															
Natural-Hip LD stem															
Natural-Hip Short Narrow Neck stem															
Natural-Hip System															
Omnicares stem															
OPTAN stem															
Orphee uncemented HA stem															
PF Primary stem															
PF Revision stem															
PFM-Revision stem															
Precedent Revision Hip system															
PYRAMID stem															
REVITAN PRIMARY stem															
REVITAN straight/curved stem <sup>3</sup>															
Segmental Proximal Femur															
Segmental TM Proximal Femur															
SF stem															
SKT stem															
SL Müller stem (cemented)															
STELCOR stem															
TCM stem															
Trabecular Metal Primary Hip Stem															
Thompson monoblock stem															
VARIALL stem (except ALLOCLASSIC VARIALL SLV stem)															
VerSys Epoch RNL Stem															
VerSys Epoch Stem															
VerSys System <sup>4</sup>															
VIRTEC stem															
VIRTEC, Long stem															
Wagner Cone prosthesis stem															
Wagner SL Revision stem <sup>5</sup>															
Weber/Stühmer and Weber Curved stem															
ZMR (Body)															

1. See "Articulating Combinations" and "Head and Stem Combinations"
3. REVITAN straight stems in 14mm diameter cannot be used with femoral heads longer than +4mm.
4. VerSys Reduced Metaphyseal (RM) stems cannot be used with femoral heads longer than +7mm.
5. Wagner SL Revision stem in 14mm and 15mm diameters cannot be used with femoral heads longer than +4mm.